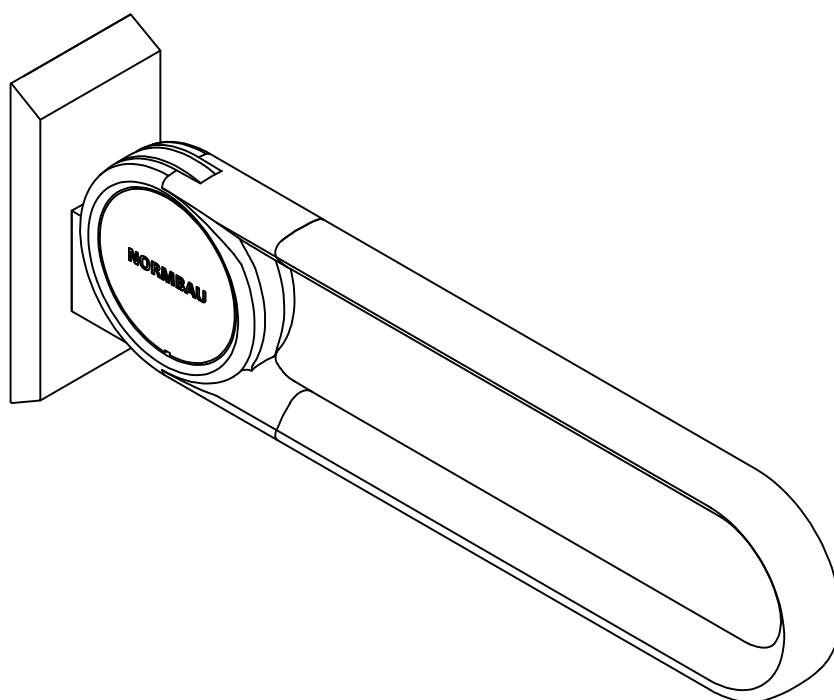
NORMBAU

PASSION FOR CARE

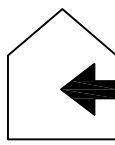
Cavere[®] Care

- (DE) Montage- und
Gebrauchsanleitung
- (GB) Fixing instructions and
directions for use
- (FR) Instructions de
montage et d'utilisation
- (NL) Montage- en gebruiks-
aanwijzing



(DE) TÜV geprüft
(GB) TÜV tested
(FR) Testé par TÜV
(NL) TÜV-gekeurd

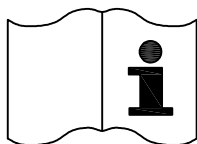(DE) Nur für den Innenbereich
(GB) For interior only
(FR) Uniquement pour un usage intérieur
(NL) Alleen voor gebruik binnenshuis

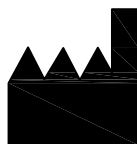(DE) Montageanleitung beachten
(GB) Follow installation instructions
(FR) Respectez les instructions de montage
(NL) Neem de montagehandleiding in acht

(DE) Hersteller
(GB) Manufacturer
(FR) Constructeur
(NL) Fabrikant

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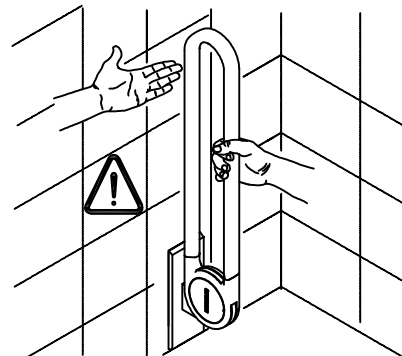
(GB) Warnings:

To be fitted by a professional and using only NORMBAU assembly kits or appropriate fixing material. An E-switch must be installed and connected by an electrician. If not professionally fitted or are used inappropriately, NORMBAU accepts no liability for any resulting losses or damage. The warranty period under current legislation is 2 years.

Before fitting, the wall construction must be checked to determine the correct fixing material.

The max. permissible load for the product is:

Cavere folding wall grab 7447... 100 kg

Cavere wall grab 7467... 100 kg

Do not use the products if you are heavier than the max. load, as this could cause permanent damage to the product. In this case the product must be checked before further use!

Caution risk of injury:

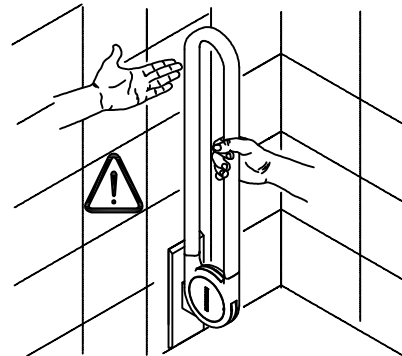
When lifting the grab up, take care not to place your hand between the grab and the wall.

Intended use according to Directive 93/42 EEC:

NORMBAU products are designed for use in closed rooms such as bathrooms and toilets. The seats provide support for people with or without physical disabilities. Caution: Exposure to the sun or washing with hot water can result in temperatures greater than 41°C, which can lead to skin damage/burning.

The materials used by NORMBAU do not pose any toxic risk.

NORMBAU products are made of recyclable materials and are designed for re-use.

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











(GB) Installation kits only for: cavere folding wall grab; cavere wall grabs

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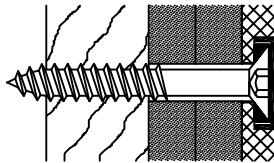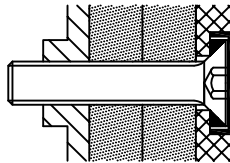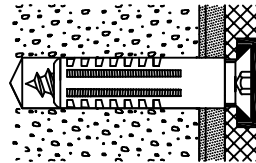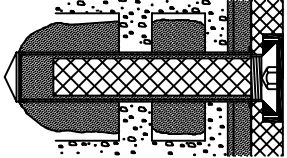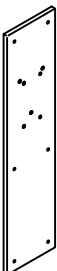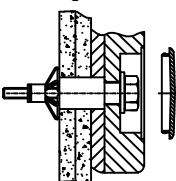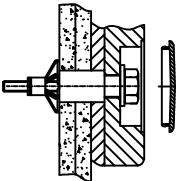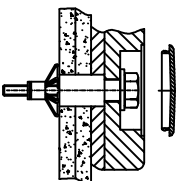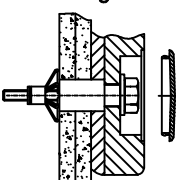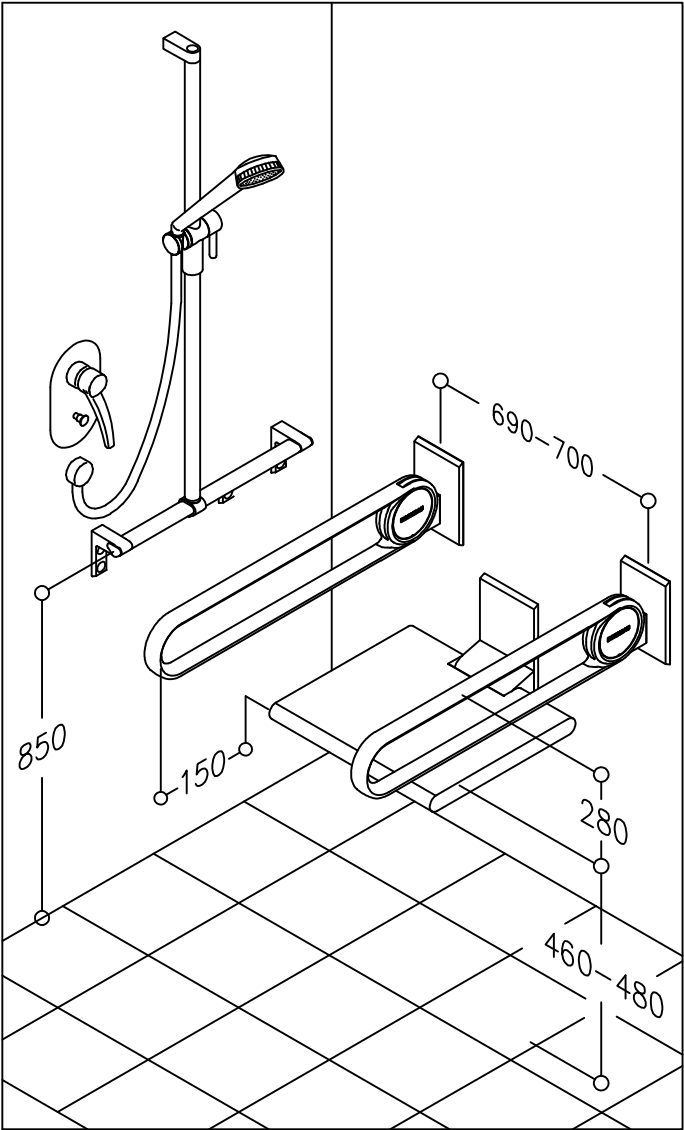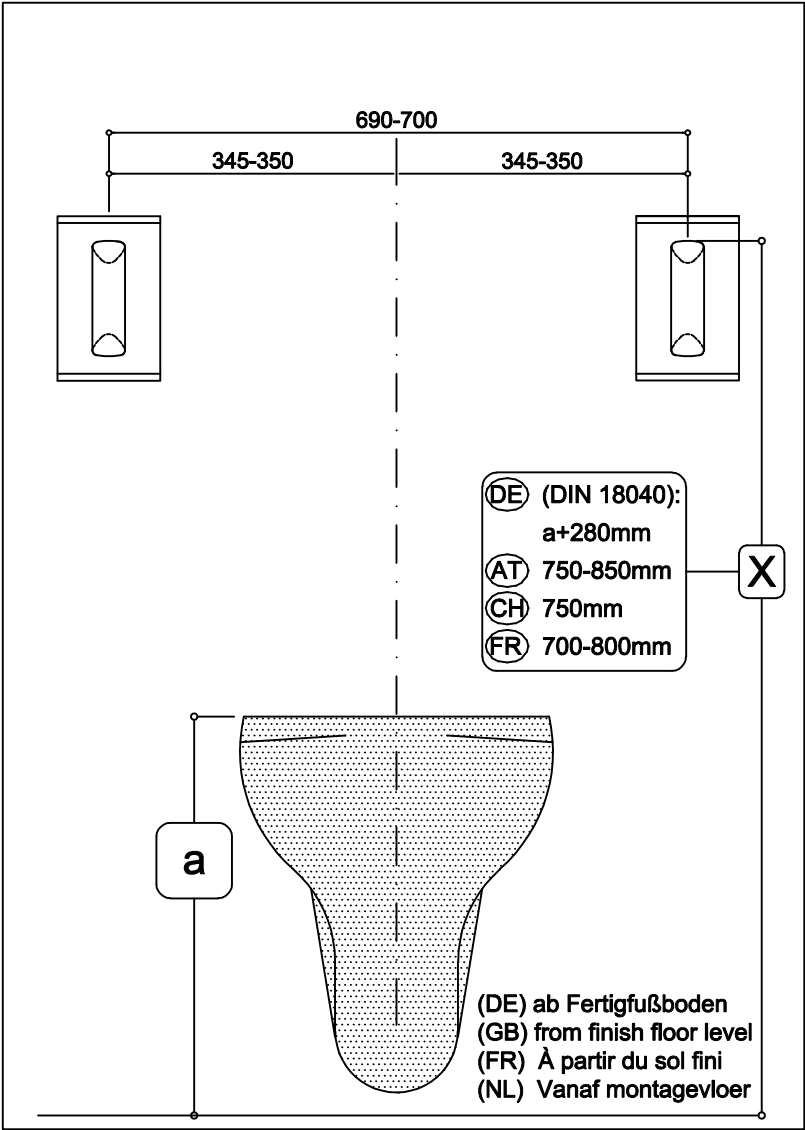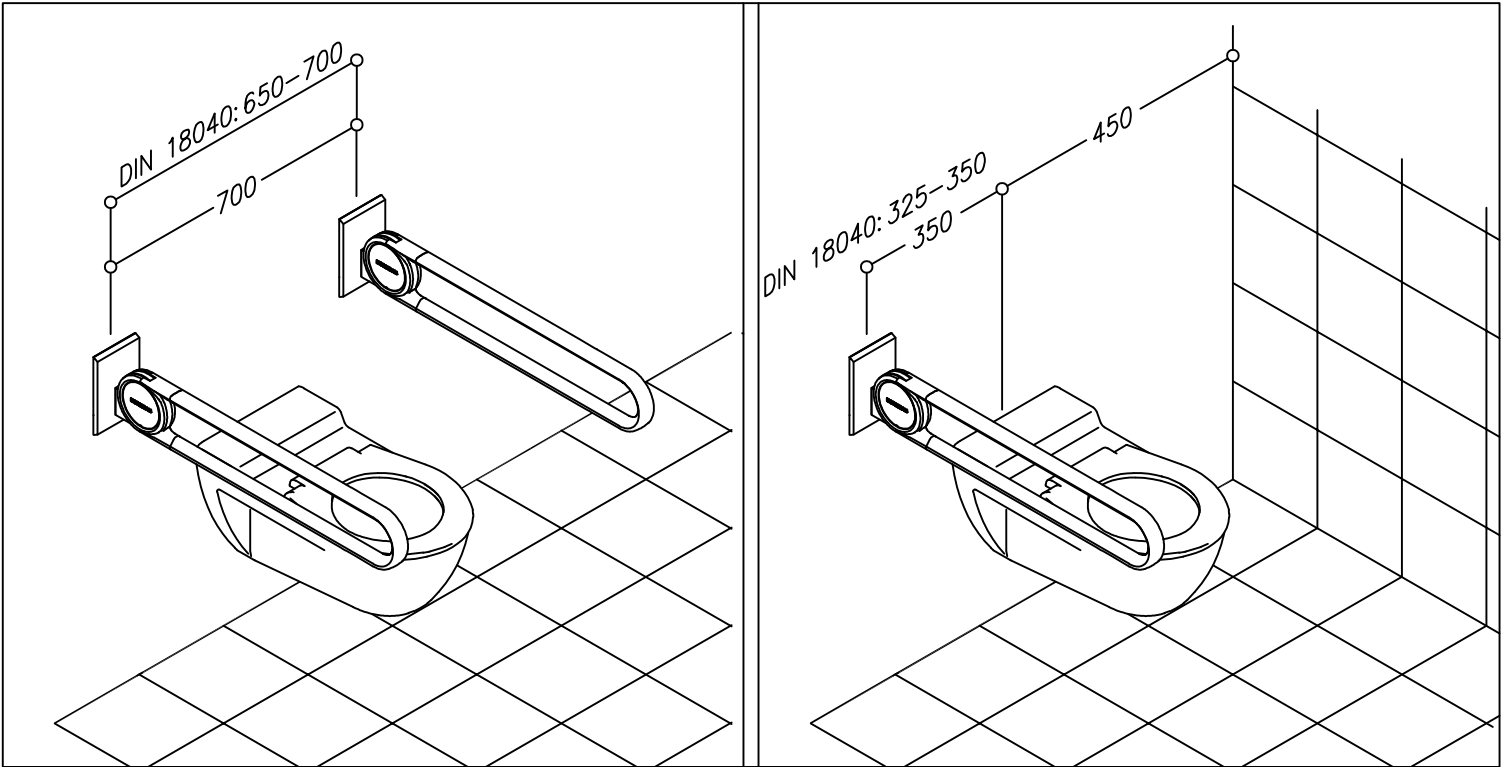
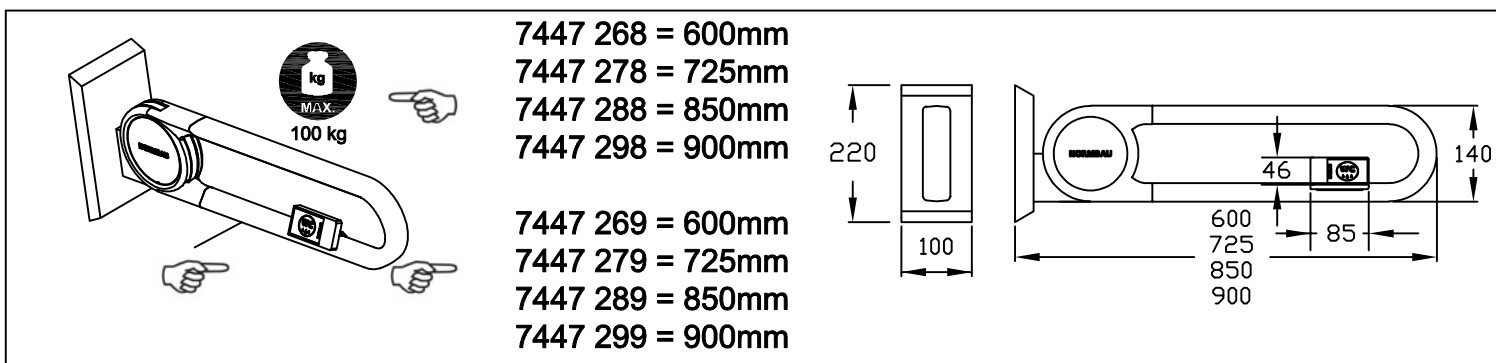
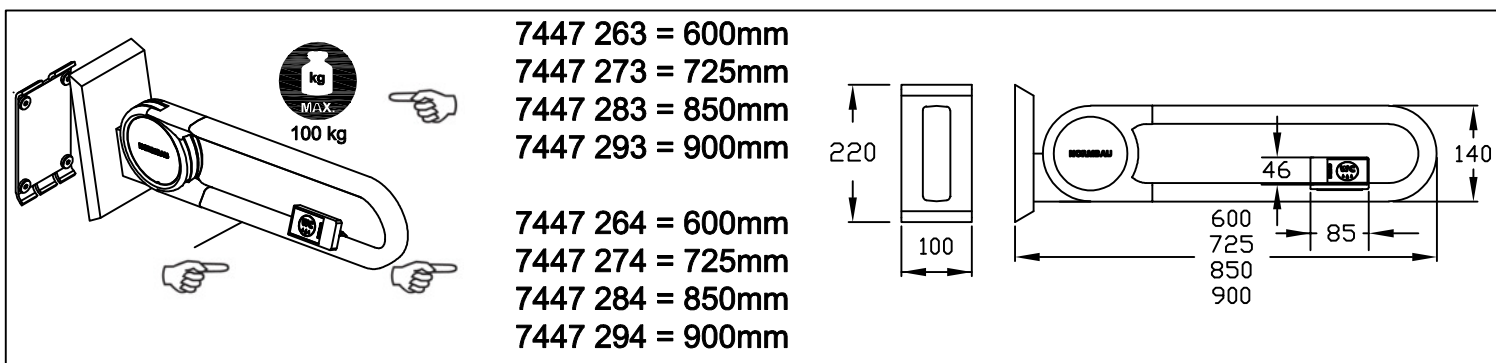
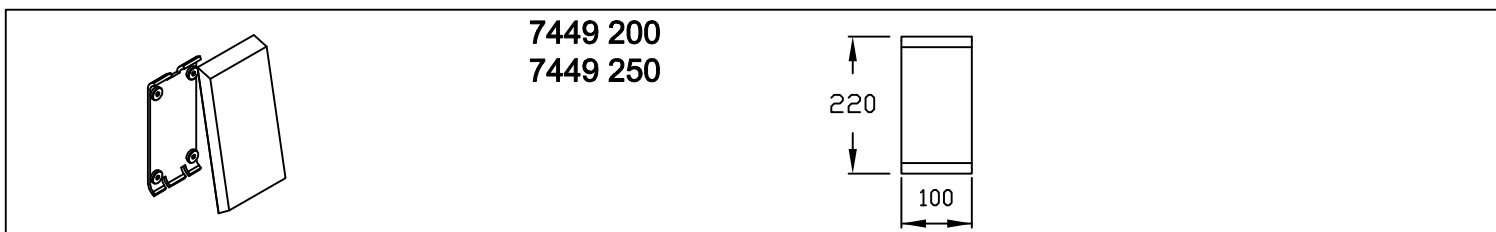
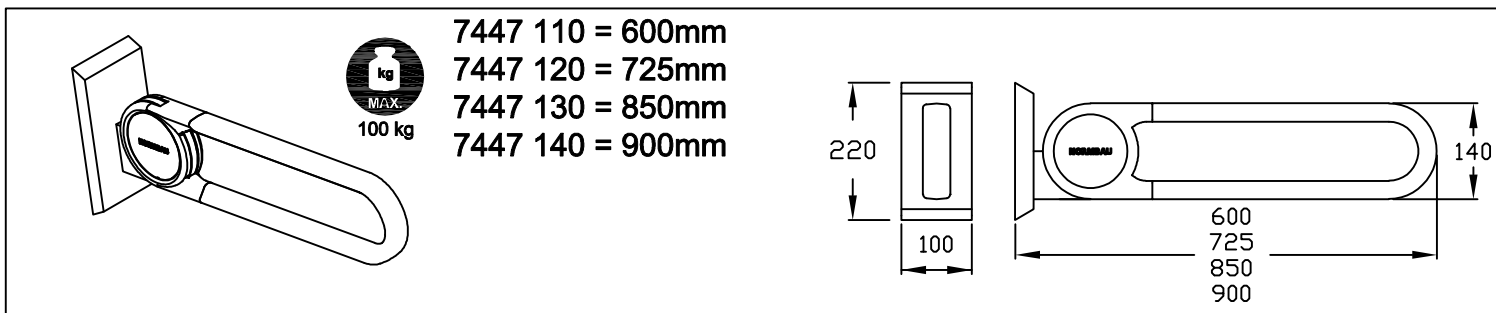
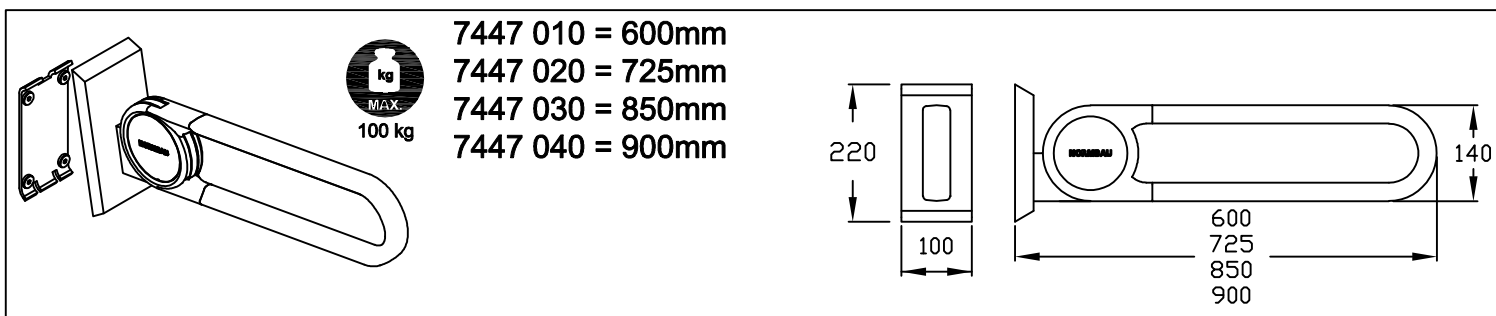
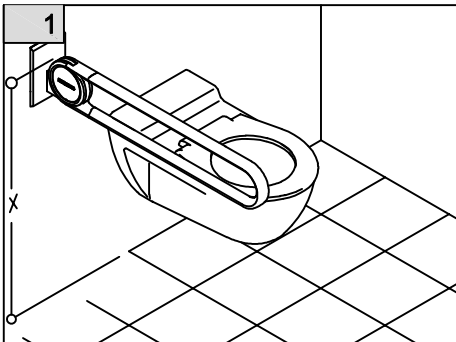
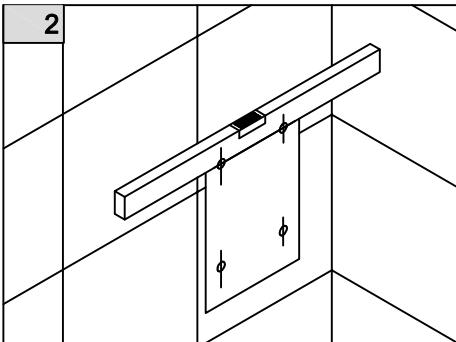
Align template (EDV-Nr. 8214 317) and mark drilling positions. Drill holes for the anchors and insert plugs.

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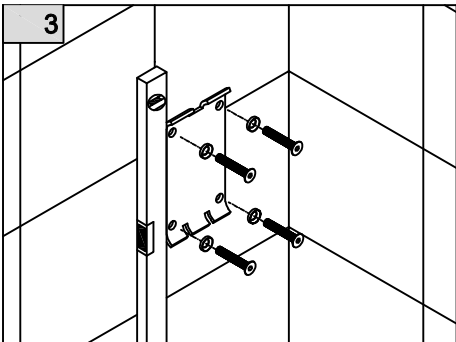
Fix base plate with screws, align vertically and tighten screws.

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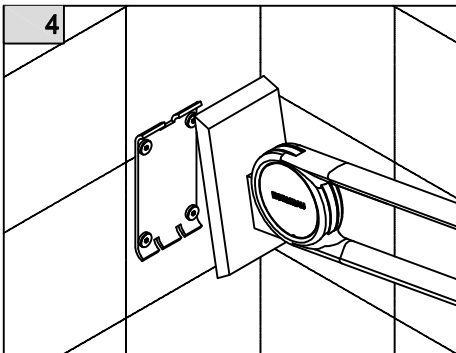
(GB) Hook lift-up support rail into the base plate.

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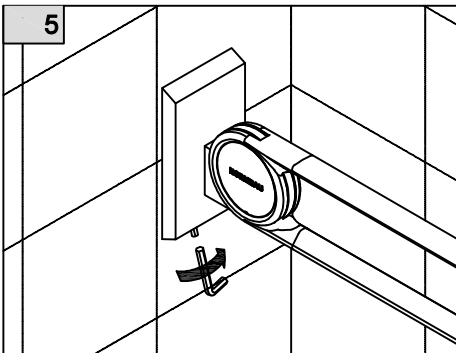
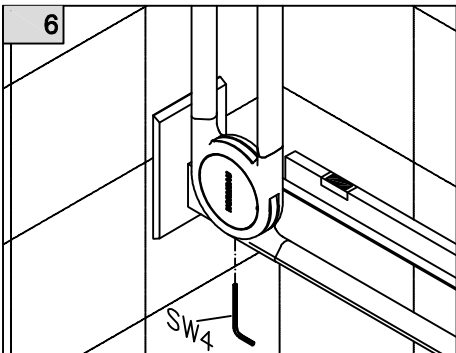
Align lift-up support rail horizontally from bottom by turning the Allan key SW4. clockwise + counter-clockwise - Adjustment of automatic built-in stop on page 21.

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
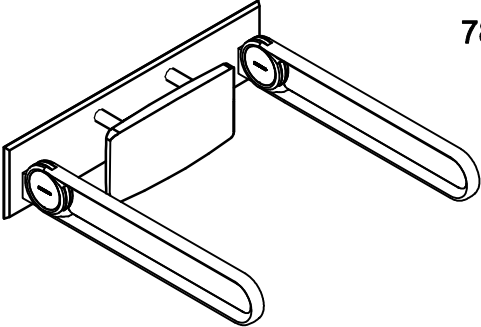
7841 320
7841 321

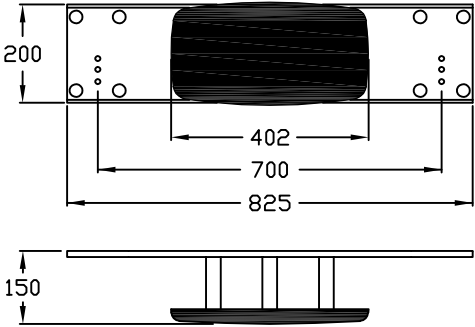100 kg

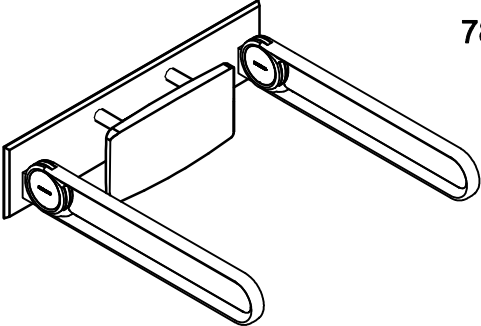
Top view dimensions: 200 (height), 402 (width of backrest), 700 (width of base), 825 (total width).
Side view dimensions: 150 (height of base).


7841 322
7841 323

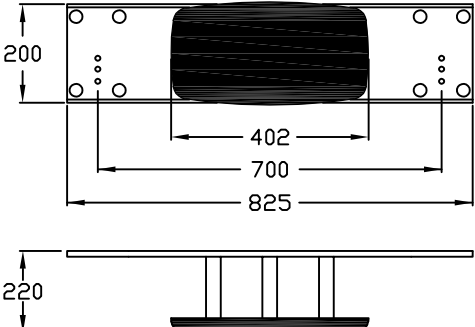100 kg

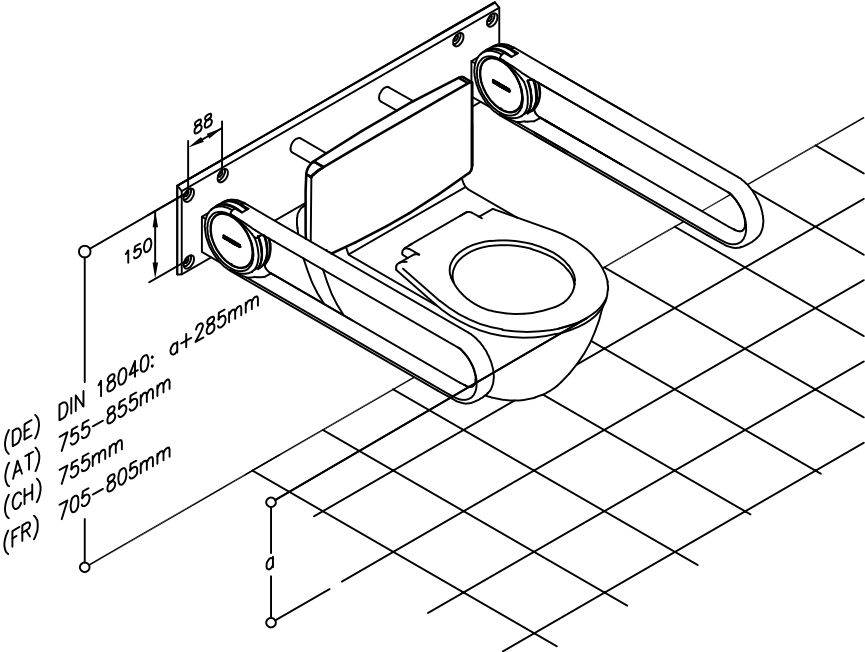
Top view dimensions: 200 (height), 402 (width of backrest), 700 (width of base), 825 (total width).
Side view dimensions: 220 (height of base).

Isometric view showing the assembly of the backrest on a toilet. Dimensions include 88mm for the backrest height, 150mm for the base height, and a+285mm for the distance from the wall to the backrest. The distance from the wall to the toilet is labeled 'a'.

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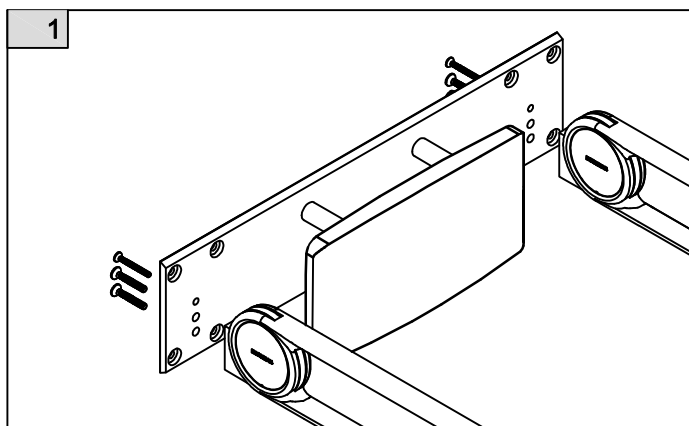
(GB) Screw lift-up support rails on fixing plate.

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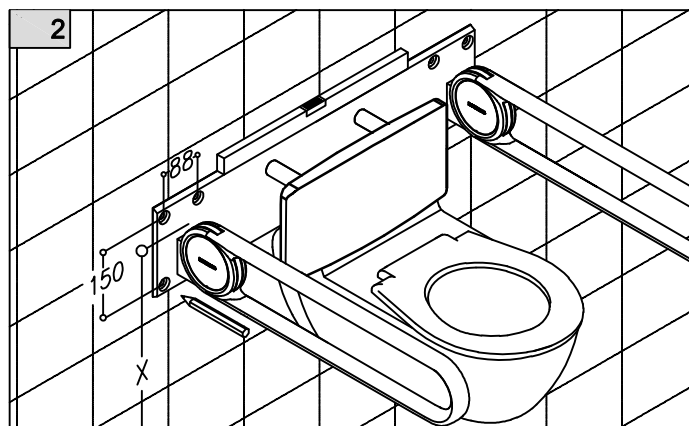
Set fixing height X, see p. 8
Align template (EDV-Nr. 8214 284) and mark drilling positions. Drill holes for the anchors and insert plugs.

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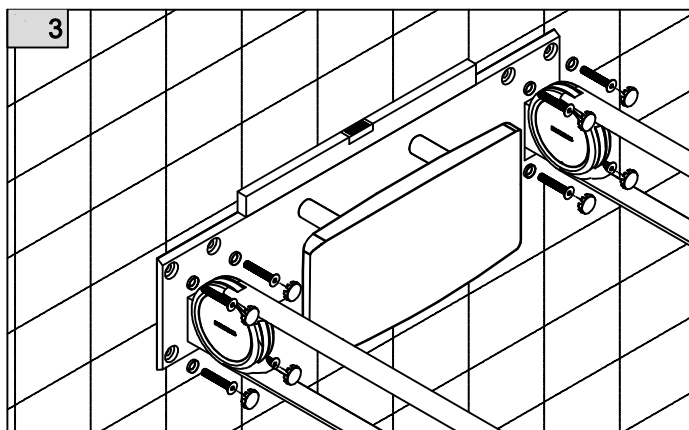
(GB) Fix backrest with screws, align vertically and tighten screws.
Press on cover.

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
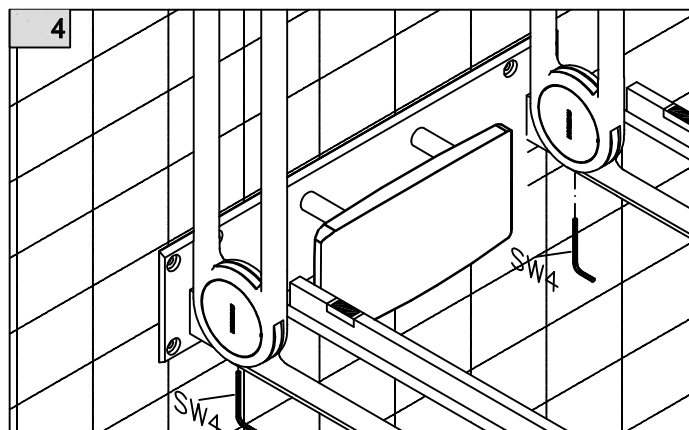
Align lift-up support rail horizontally from bottom by turning the Allan key SW4.
Clockwise +
counter-clockwise -
Adjustment of automatic built-in stop on page 21.

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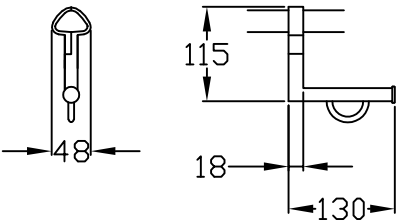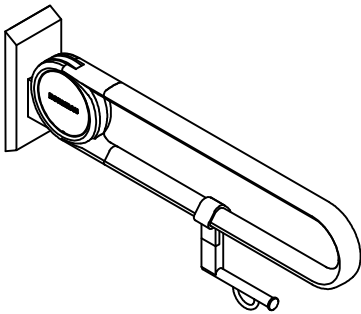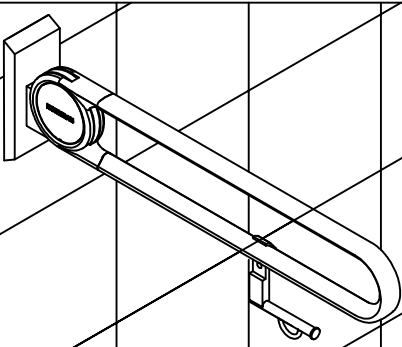
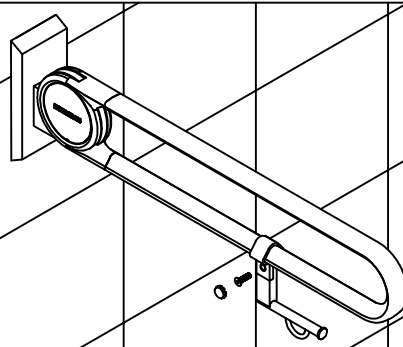
(GB) Connect paper roll holder to lift-up support rail.

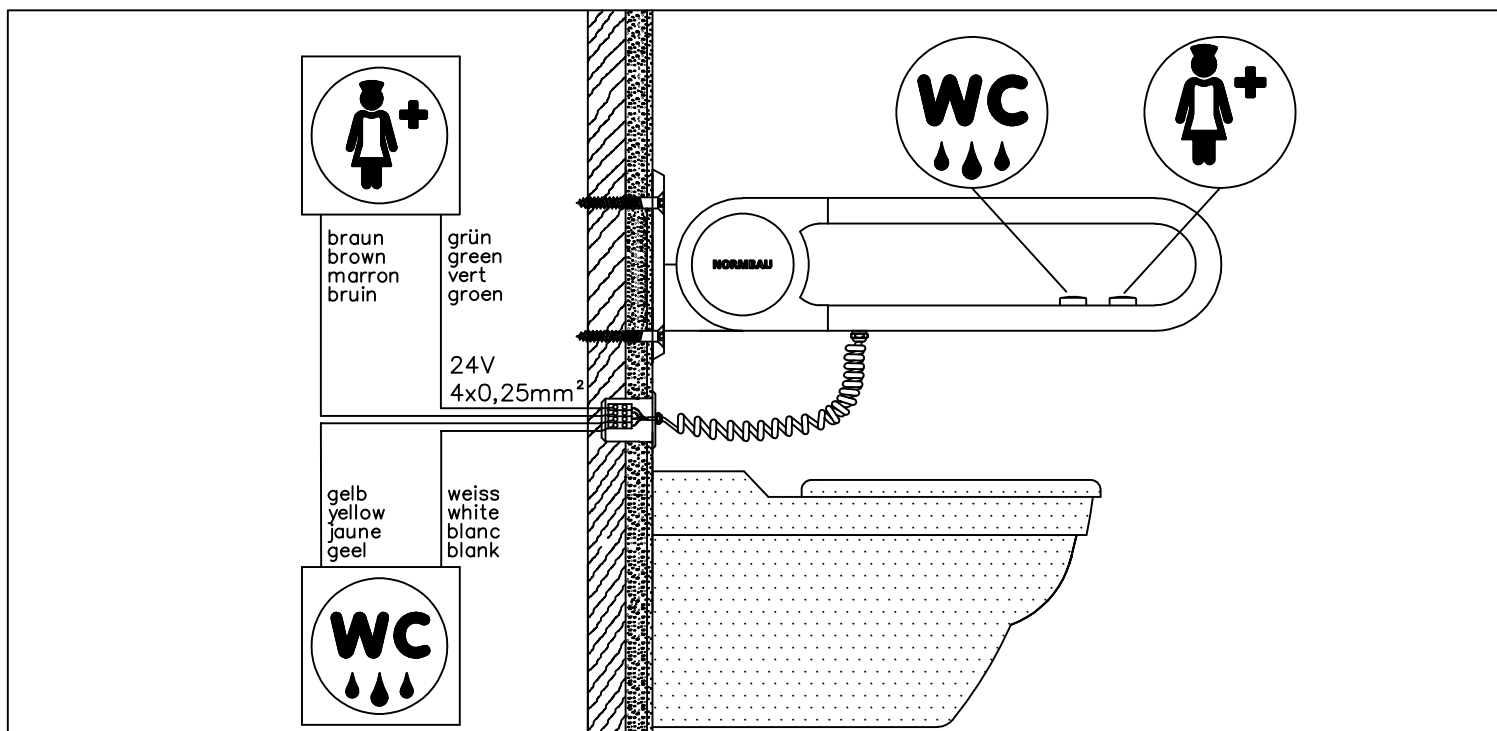
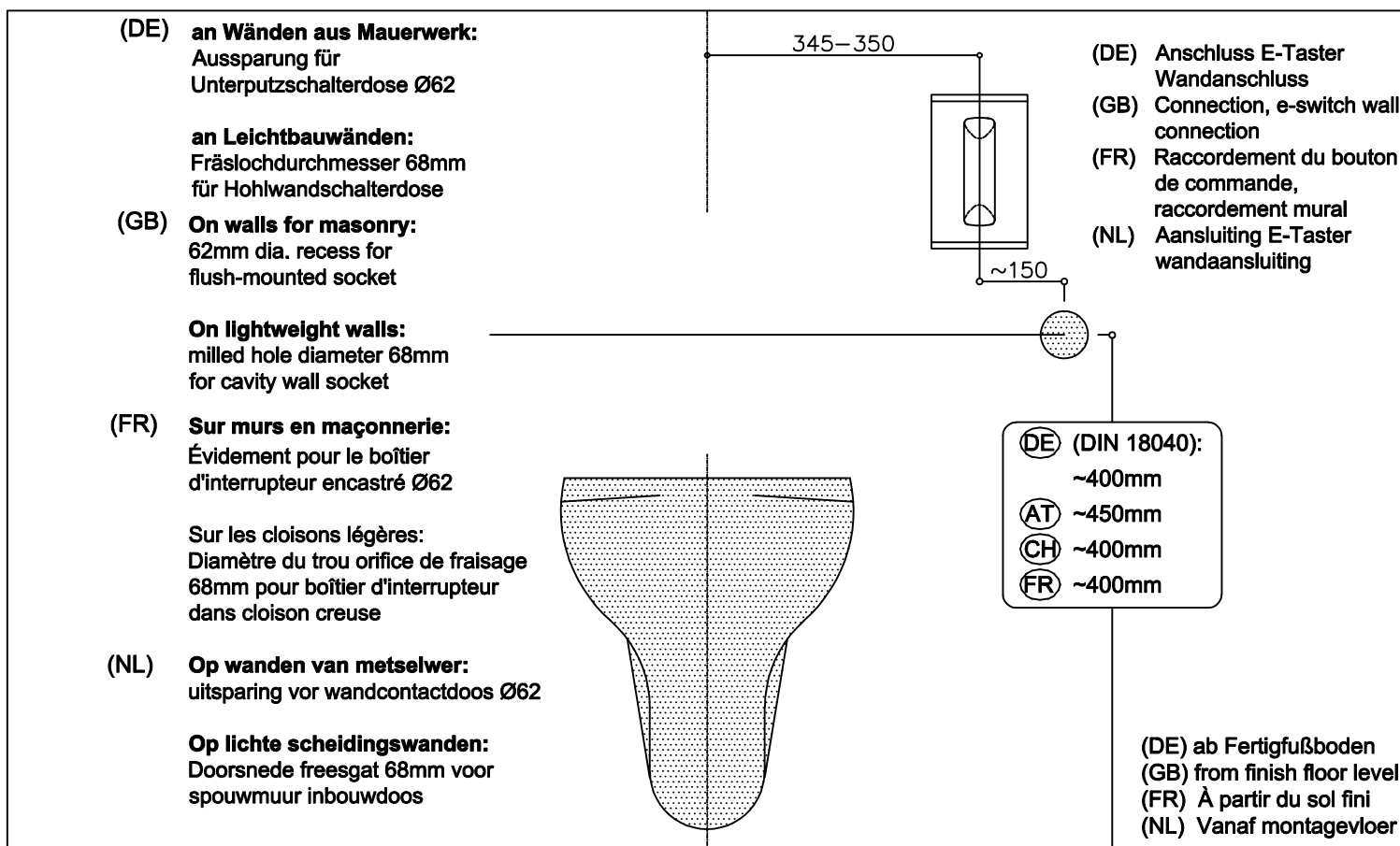
Screw paper roll holder and holding device together. Press in cover cap.

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
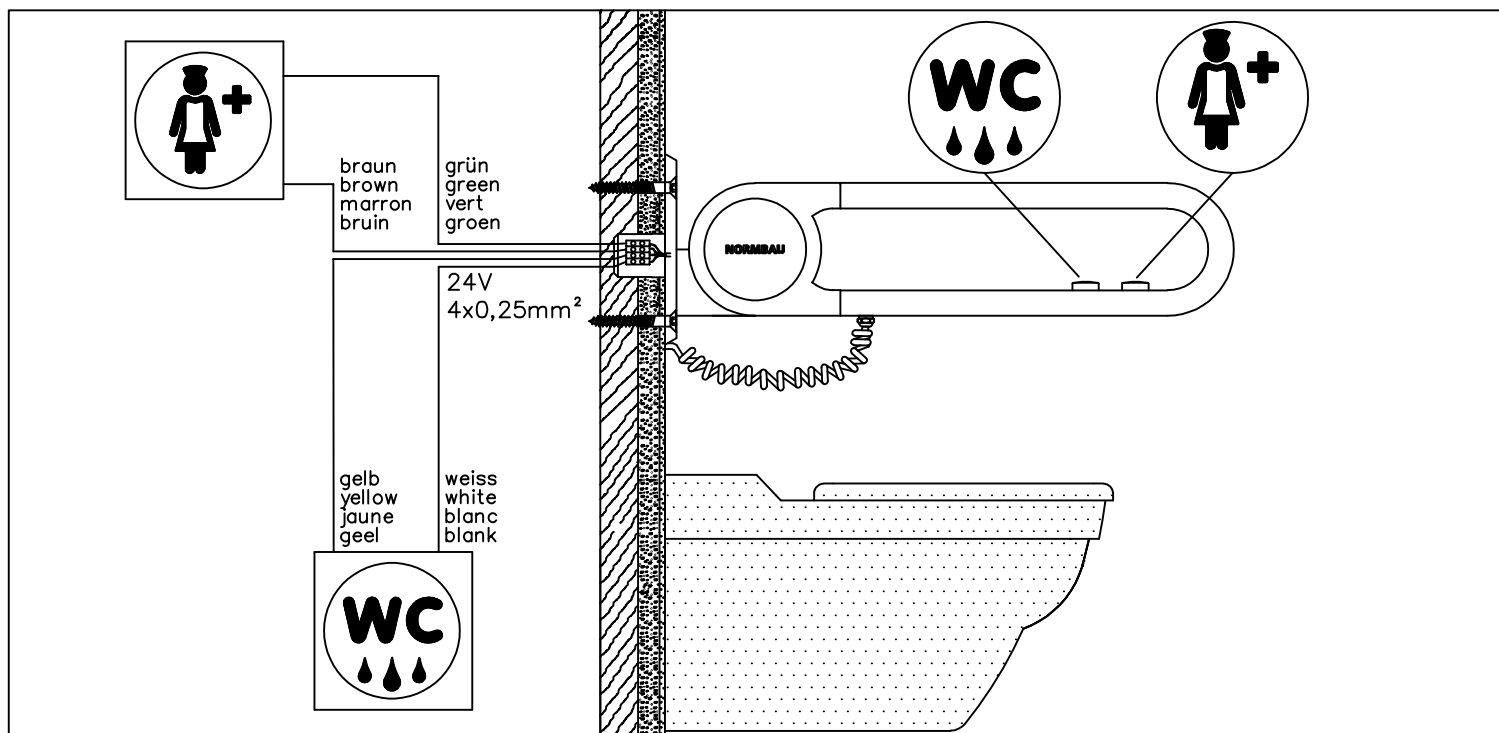
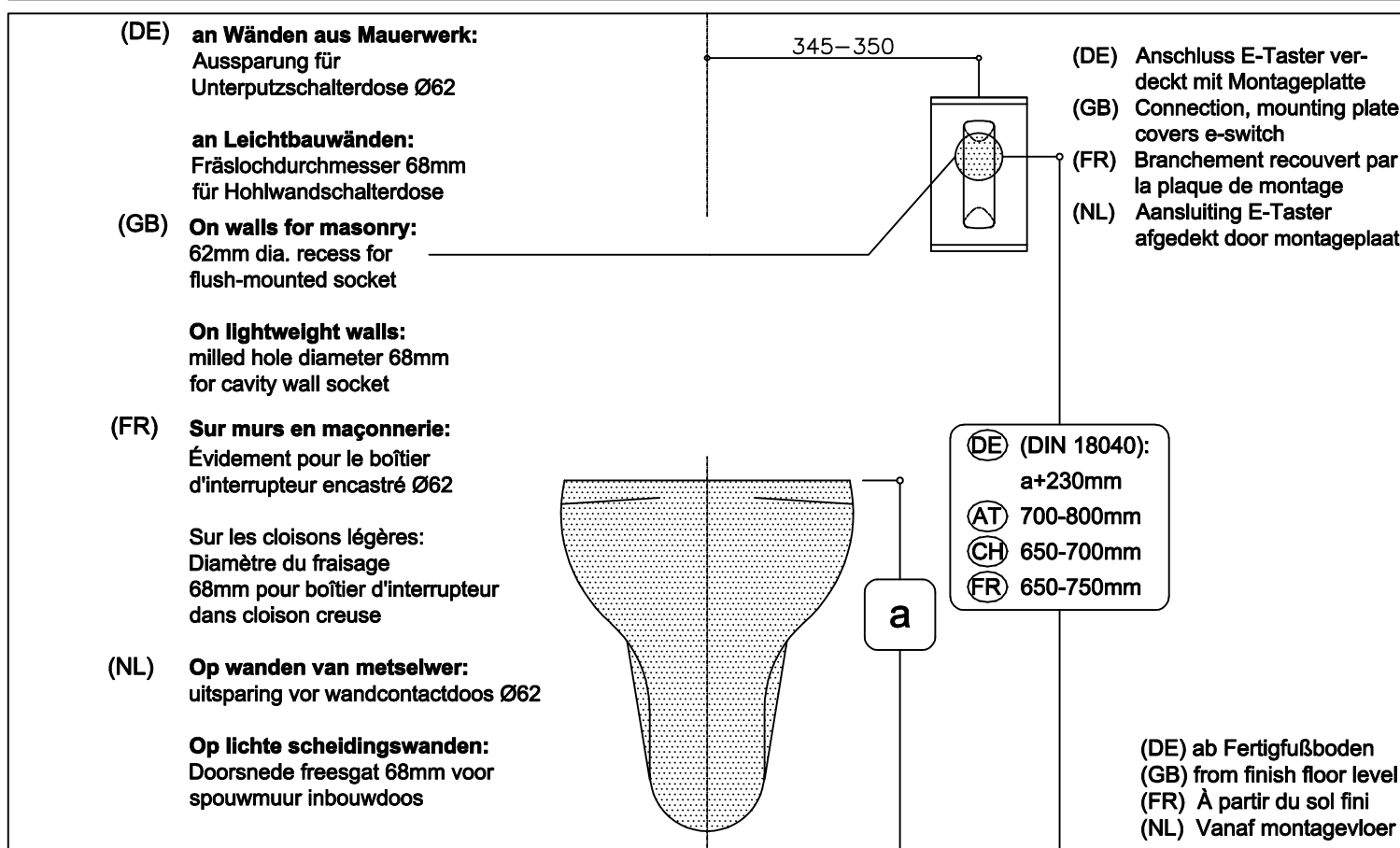
(GB) If the e-switch is connected as a call button on an existing intercom according to VDE 0834, a cable relief must be fitted in the junction box. The button is supplied as normally open and can be modified on site with a soldering iron to normally closed. A minimum of protection type IP X4 must be provided for the junction box (junction box Up6). The permitted maximum voltage for WC controls or for emergency call systems is 24 V.

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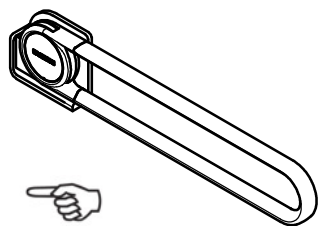
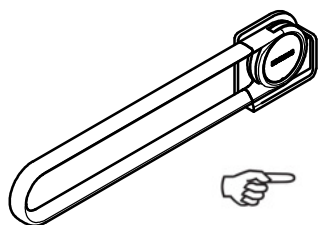
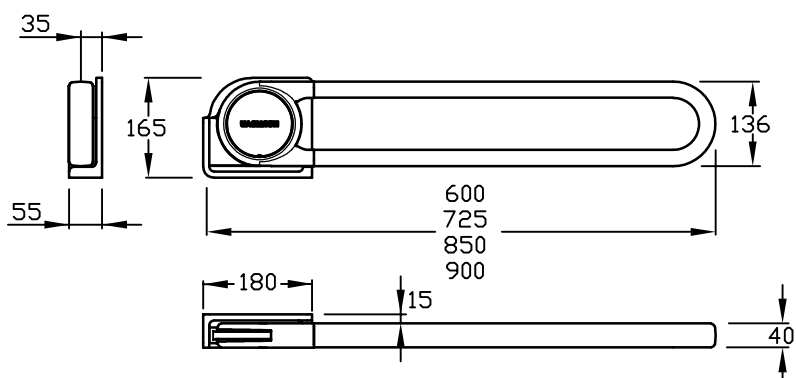
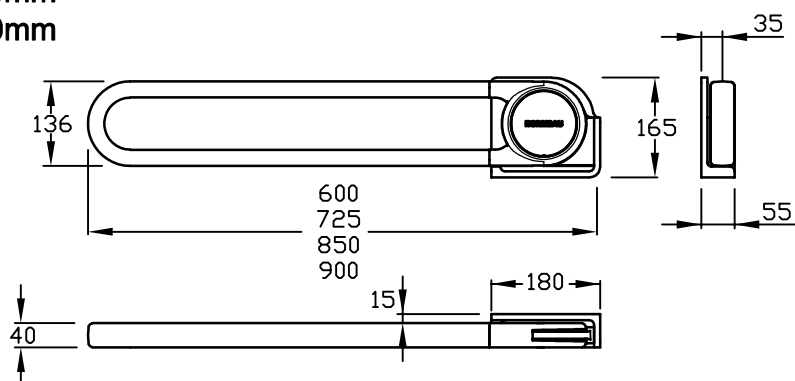
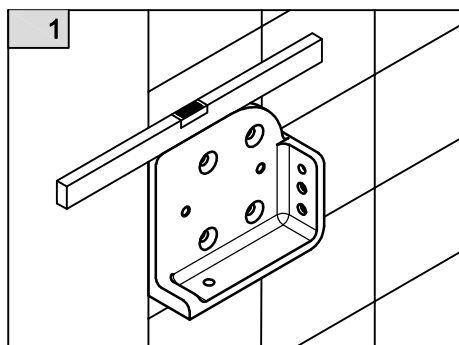
(GB) Align the side mount and mark drilling positions. Drill holes for the anchors.

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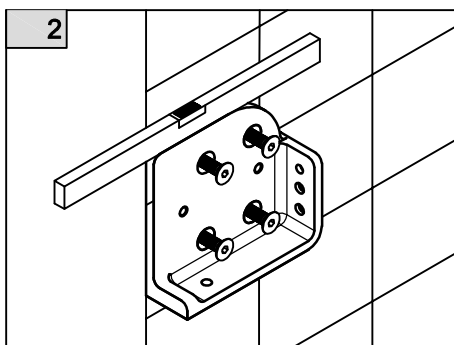
Fix the side mount with screws, align the side mount and tighten screws.

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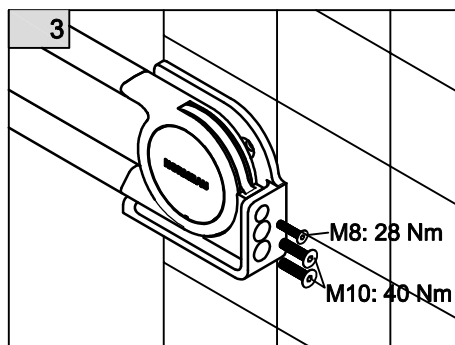
Fix lift-up support rail with screws and tighten screws with the specified torque. First tighten the bottom screw.

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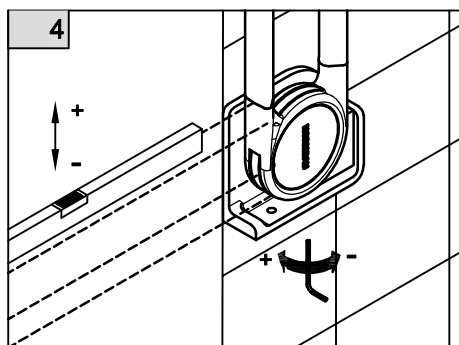(GB) Align lift-up support rail horizontally from bottom by turning the Allan key SW4. clockwise +, counter-clockwise -.

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(GB) Maintenance:

Cavere folding wall grabs have maintenance-free slide bearing technology, with an adjustable brake (see installation instructions: setting the braking force). Cavere wall grabs are maintenance-free.

Cavere grabs and angular grabs are maintenance-free.

Cavere shower head holders have stepless and smooth height and angle adjustment, with maintenance-free self-adjusting clamping force.

Cavere seats have a maintenance-free brake, so that no maintenance is required.

If the product suffers serious damage, please inform the specialist supplier or NORMBAU. Do not continue to use the damaged product!

Care instructions:

NORMBAU powder-coated products have a smooth surface that is easy to clean. To preserve the high quality powder coating of our products for many years, it is important to clean and care for the products appropriately. Minor soiling can be removed with a sponge dipped in water to which a neutral cleaning product (e.g. washing-up liquid) has been added. After cleaning, rinse immediately using clean, cold water. A chamois leather or microfibre cloth can be used to remove smears or any droplets of water. Abrasive or strong acidic/alkaline cleaning agents should never be used. Heavy soiling on metallic coatings should be removed as soon as possible, as dried-on dirt can often only be removed using abrasive cleaning agents, which may cause damage to the powder coating or result in a colour change. Cleaning agents should not be left to act for more than an hour. If necessary, the cleaning process may be repeated after at least 24 hours. Greasy, oily or sooty substances can be removed using alcohol-based or bathroom cleaners. Adhesive residues can be removed in the same way. However no solvents should be used for cleaning. If disinfection is necessary, normal disinfectants can be used in appropriate concentrations. Additional, preservative cleaning can be undertaken once or twice a year. When using cleaning products and disinfectants, manufacturers' instructions must always be followed. Owing to the properties of powder coatings and the risks of a change in colour and/or effect, we recommend always checking the suitability of cleaning products and disinfectants.

The upholstered seats should preferably be treated with mild over-the-counter cleaning agents and disinfectants with low alcohol content, such as the compound Microx (cleaning agent and disinfectant), Meltan (cleaning agent) from ECO Point in Schüttorf, Terralin and Meliseptol as surface disinfectants. Products containing iodine may cause staining. Care products containing dyes should not be left on for long periods. Avoid exposing upholstered seats to temperatures over 80°C!

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NORMBAU GmbH